

EasiCollect Plus

Collection Protocol




1



BUCCAL CELL COLLECTION

Gloves should be worn while handling the device and during collection. Ensure that the foam head is coated with saliva, by dampening the foam head under the tongue, to ensure good sample transfer. After dampening, rub the collector on the inside of each cheek for 15 seconds.


2

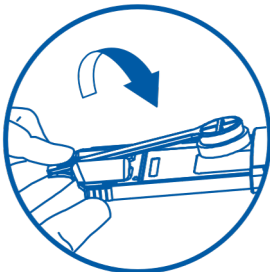

FTA CARD APPLICATION

Close the buccal collector so it comes into contact with the paper, which will allow for transfer of the sample to the collection card.


3

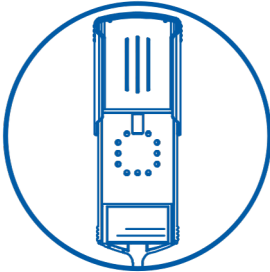

DRYING

Holes in the cover of the device allow for drying during storage and transport.

4

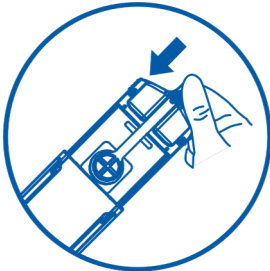

STORAGE/TRANSPORT

Close the device cover to ensure a secure and well-protected card during transport. When closed properly you should hear a click and the arrows on the device should align. Complete the identification information on the rear of device.



Scan here to
watch a
video of the
collection
procedure!

